## ***TRETS REH***

**Gebrauchsanleitung**  
**User's Manual**

13

14

**Komponenten:**

**Components:**

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1. Main frame
2. Frame quick-adjust assembly
3. Pedal with Toe Clips and Straps
4. Reflectors
5. Fork quick-release
6. Front boom
7. Headlight
8. Handlebar
9. Brake levers with parking brake
10. Grip
11. Grip shifter
12. Seat
13. Rear light with reflector
14. Seat adjustment
15. Serial number / bike coding
16. Protection disc with reflector- stripes for radius arms (appropriate according to StVO §67 para.7 and ECE-formalities)

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**Contents**

**Options and Accessories..... 4**

**Description ..... 5**  
 What is the Trets Reh? .....5  
 Intended use.....5  
 Indications for use .....5  
 Contraindications .....5  
 Adaptability to individual needs.....5  
 Multiple users .....5

**Safety information ..... 6**  
 Read User's Manual .....6  
 Symbols used for notes and warnings.....6  
 Before your first ride.....6  
 Observing traffic regulations .....6  
 Protection against environmental influences .....6  
 Cleaning and disinfection .....6  
 Applicable standards and regulations .....6  
 Tipping and overturning.....7  
 Braking.....7  
 Suitable clothing and helmet .....7  
 Quick-release mechanisms .....7  
 Risk of injury from rotating parts .....7  
 Crushing hazard .....7  
 Risk of flammability.....8  
 Pedals.....8  
 Safety belt .....8  
 Being visible for others.....8  
 Riding at night.....8  
 Technical condition .....8  
 Load carrying capacity of the Trets Reh .....8  
 Serial number and bike coding.....8

**The first ride..... 9**

**Adjusting your Trets Reh ..... 10**  
 Checking the seat position .....10  
 Adjusting the frame length.....10  
 Adjusting the handlebar height/angle .....11  
 Adjusting the brake levers to hand size.....12  
 Adjusting the seat angle.....12  
 Adjusting the safety belt.....12

**Riding your Trets Reh..... 13**  
 Mounting and dismounting .....13  
 Attaching the flag .....13  
 Correct tire pressure .....13  
 Shifting .....14  
 Shifting gears with the twist shifter .....14  
 Brakes .....14  
 Transporting luggage.....15  
 Fender-stay pouches.....15  
 Parking the Trets Reh upright .....16  
 Transporting the Trets Reh .....16  
 Packing the Trets Reh in a cardboard box .....17

Operating the lighting system .....17

**Maintenance and repair ..... 18**  
 Maintenance intervals .....18  
 Checking wheels for lateral/radial true.....19  
 Checking tires .....19  
 Changing tires or repairing a flat.....19  
 Adjusting the fenders.....20  
 Checking disc brakes .....20  
 Checking the chain .....21  
 Cleaning and oiling the chain.....21  
 Checking the chain tube.....22

**Appendix for dealers..... 23**  
 Technical data .....23  
 Tightening torques.....23  
 Technical data of the accessories.....24  
 Accepting delivery .....26  
 Unpacking and checking contents.....26  
 Assembling the Trets Reh.....26  
 Removing the rear wheels.....27  
 Mounting the the rear wheels .....28  
 Removing the internal gear hub .....29  
 Installing and adjusting the internal gear hub.....29

**Replacement parts..... 30**  
 Trets Reh, complete .....30  
 8-speed internal gear hub .....31  
 Steering tube.....32  
 Wheel components, left.....33  
 Wheel components, right.....34  
 Frame quick-adjustment.....35

**Accessories ..... 36**  
 Holder for walking sticks .....36  
 One-Hand Controls.....37  
 Crank Shortener .....38  
 Pedal Pendulum .....39  
 Spezial Pedals .....40  
 Special Pedals with Calf Support.....41  
 Pedal with Toe Clips and Straps .....42  
 Lap and Shoulder Belts.....43  
 Fixed gear .....44  
 Direction indication lights LED .....45  
 One-Sided Leg Rest.....46  
 Push Rod Trets.....47

**Long-term storage and disposal ..... 48**  
 Preparing the Trets Reh for long-term storage, and for use after long-term storage .....48  
 Preparing the Trets Reh for disposal .....48

**Conditions of warranty and liability ..... 49**

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### Trets Reh

The children's trike with 8-speed gear hub

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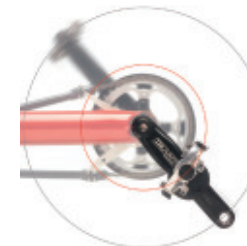
LED-Blinkanlage  
LED blinker system

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## Description

### What is the Trets Reh?

The Trets Reh is a recumbent children's trike and an approved medical device as per the European Directive 93/42/EEC (Medical Device Directive). The trike is driven by leg power via the pedals, which are specially designed to prevent the feet from slipping out when riding. The transmission is a fully enclosed 8-speed hub gear system that drives one rear wheel. The padded seat has been specially designed and constructed with flexible materials to cushion the ride and offer maximum comfort. The Trets Reh has a reliable brake system with two independently operated disc brakes.

### Intended use

The trike is intended for use on paved roads and paths. It can also be adapted for use by adults of very short stature. The required technical adjustments, however, must be performed by an authorized retailer. Bikes and trikes should not be ridden in traffic unless they have been equipped with the accessories required by the applicable traffic regulations of the country in which they are being used. The Trets Reh comes with a lighting system and all safety features required by the German road traffic regulations (StVZO). In Germany, children under the age of eight may only ride the Trets Reh on sidewalks under adult supervision, even if the adjacent road has a bike path. Children ten years of age and older may ride in road traffic under adult supervision. Children from eight to ten years of age may ride on sidewalks and bike paths. Please observe the applicable regulations in other countries.

The Trets Reh is not approved for off-road use or racing.

### Indications for use

The Trets Reh has been designed for children with neuromuscular disorders, such as cerebral palsy or muscular dystrophy, who – owing to special physical conditions – are unable to ride standard children's bicycles or vehicles, even with training wheels. Other indications include coordination disorders, balance disorders, and restricted movement associated, for example, with spasticity, paraplegia, dysmelia, amputations, etc.

### Contraindications

The Trix is not suitable for children with visual impairment.

### Adaptability to individual needs

The Trets Reh can be equipped to accommodate riders with various physical impairments. The following are just a few examples of available accessories: single-hand controls, crank shorteners, special pedals with or without calf support, pedal pendulums, and holders for walking sticks.

### Multiple users

Trets Reh trikes that have been used in accordance with their intended use can be reused by a new rider after the trike has undergone an inspection by an authorized retailer.

### Protection against environmental influences

The Trets Reh is constructed using special materials that protect it from corrosion: powder-coated

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aluminum frame parts, plastics, and stainless steel.

### Cleaning and disinfection

The handlebar grips and seat can be cleaned and disinfected using household cleaning agents and disinfectants. For more information, see the section “Cleaning and disinfecting your Trets Reh” on page 23.

### Applicable standards and regulations

The Trets Reh has been designed and manufactured in compliance with the following regulations and standards:

European Directive 93/42/EEC (Medical Device Directive)

## Safety information

### Read User's Manual

Read the User's Manual carefully and familiarize yourself with the construction and function of the vehicle in practice. Involve your child in this process and share with her the information you have read. Be sure that your child has understood all safety-related functions.

### Symbols used for notes and warnings

#### **Caution!**

This symbol indicates that there is a risk to life and health if the respective instructions are not followed.

#### **Note:**

This symbol is used for information and tips on the handling and use of the vehicle.

### Before your first ride...

Your authorized retailer has assembled your new Trets Reh and adjusted it to the specified size requirements of the child. Please refer to the section “Adjusting your Trets Reh” on page 9 if you need to make adjustments for another rider. Before the first ride, please take some time to familiarize yourself with the vehicle; give the child a thorough explanation of all important functions, such as gear-shifting, braking, etc., and discuss potential risks and dangers. We recommend letting the child familiarize himself with the vehicle's handling in a quiet, open area away from traffic before riding on public roads.

### Observing traffic regulations

The Trets Reh comes with a lighting system and all safety features required by the German road traf-

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fic regulations (StVZO) and may only be used in road traffic with this equipment. Be sure to study the applicable regulations and consult your authorized retailer before making any changes to the lighting system.

Traffic regulations are different in every country. The Trets Reh and its equipment must be in compliance with the regulations of the country in which it is used.

### Tipping and overturning

Recumbent trikes have a low center of gravity. This allows for cornering at higher speeds. If the rider's speed is too high, however, the centrifugal force can cause the Trets Reh to skid or even tip over, which could cause injury to the young rider or others. The likelihood of tipping over while cornering is increased by hectic steering movements and braking.

Special care must also be taken when stopping on inclines. The Trets Reh could roll backwards and, if the brakes are applied abruptly, even overturn. Be sure to discuss this danger with the young rider. A suitable helmet can protect the rider from head injury.

### Braking

The disc brakes of the Trets Reh are very powerful and have good modulation. However, it is still possible for the wheels to become locked. Before his first ride in public traffic, the rider should take time to familiarize himself with the braking force by test-riding the trike in an open area away from traffic.

### Suitable clothing and helmet

The Trets Reh rider should never wear loose clothing that could get caught in the wheels or gears. Beware of scarves and clothing strings/cords that hang down. A bicycle helmet can protect the rider from head injury in the case of accidents.

### Quick-release mechanisms

Prior to every ride, be sure to check whether all quick-release mechanisms are securely locked. Riding with loose or open quick-release mechanisms can lead to accidents with life-threatening injuries.

### Risk of injury from rotating parts

The wheels are equipped with spoke guards that eliminate the risk of spoke-related injury. However, there are other rotating parts that pose hazards. The Trets Reh rider should be advised never to touch the disc brakes or gears, which could cause severe injuries. Never engage the dynamo when the trike is in motion. Be sure to discuss these dangers with the Trets Reh rider.

### Crushing hazard

The Trets Reh has been designed with numerous adjustment sites to make it adaptable to many different riders. Therefore, it was not possible to completely avoid joints that pose a crushing hazard. To prevent injury, be careful not to get your hands or feet caught or pinched in the trike's parts when riding or adjusting the trike. Be sure to discuss these dangers with the Trets Reh rider.

### Risk of flammability

The seat cover is made of a flammable material. If exposed to an open flame, such as a lighter, the seat cover could catch fire, which could cause severe burns to the rider. Therefore, avoid any use of

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fire near the seat cover. Be sure to discuss this issue with the user.

### **Pedals**

In a recumbent position, it is especially easy for the feet to slip from the pedals. Therefore, be sure to use the toe clips and straps (slip protection) that come with the trike.

### **Safety belt**

Secure the child in his seat using the three-point safety belt. This prevents injury in the case of abrupt braking or accidents.

### **Being visible for others**

The Trets Reh is a relatively low vehicle. As a result, the trike is less visible to other road users. The Trets Reh comes with a safety flag that improves the trike's visibility to other road users. Be sure to ride with this flag at all times.

### **Riding at night**

Always use lights when riding at night or in conditions of low visibility, such as dusk and dawn. Before each ride, be sure to check whether the lighting system is functioning properly.

### **Technical condition**

Before each trip, be sure to check the condition of the trike for possible faults or damage. Take the Trets Reh to your bike shop for regular maintenance. The recommended maintenance intervals can be found in the section "Maintenance intervals" on page 19. Use only the replacement parts and accessories recommended by HASE for the Trets Reh. Consult your bike shop for more information.

### **Load carrying capacity of the Trets Reh**

The maximum permissible combined weight for vehicle, rider, and luggage is 65 kg (143 lbs). If the Trets Reh is used as a trailer, the respective weight limit is 45 kg (99 lbs). In Germany, a combined weight of 45 kg (99 lbs) is permissible for trailers with no overrun brake. Be sure to observe the regulations that apply in the country of use. All luggage must be transported on the luggage rack in luggage bags recommended by HASE (accessories) or in equivalent bags from other manufacturers.

### **Serial number and bike coding**

The serial number can be found on the end of the main frame (see illustration on the inside cover of this manual). This is the only place where bike coding can be engraved. Such coding makes it easier to locate the bicycle owner in the case of theft or loss.

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## The first ride

Before the kid's first ride on it's own, make sure that it practiced driving supervised by an adult. Especially cornering and the usage of the brakes should be taught, because there are many differences between a recumbents behaviour and the one of a normal bike.

It is possible that the rear wheel, which is inside of a sharp bend, lifts up. To avoid this effect, the driver has to learn to shift his weight to the bend's inside.

The brake handles operate the left and right rear brakes separately. Therefore, increased caution is needed when braking one-sided to avoid wheel blockage. Otherwise lateral turn followed by an overturn may occur. The driver also should cope with dosed usage of both brakes, which should be operated together due to security reasons.

High speed while cornering and on rough ground as well as extreme wheel angles can lead to tipping of the trike. Therefore thoughtful driving with caution and avoiding of dangerous situations always shall be kept in mind.

Peculiar attention is needed when mentally handicapped persons are using the recumbent. If there are any doubts in the drivers abilities, please use the optional pushing rod to keep control over that person and the bike. In those cases never let the person drive unattended or rather instruct him repeatedly about the possible dangers of the usage in traffic.

Mind the dangers of cornering with the recumbent primarily. Another important issue is that the widest part of the bike is located behind the driver. To get used to the bike's dimensions, execute practices with obstacles.

### Important security issues:

- Pay hight attention at the very first ride!
- Always leave both hands on the handlebars and keep the feet fastened in the pedals
- Test handling on rough ground to achieve better reaction to unexpected steering actions
- Practice the repond of the bike to the shifting of the body weight
- Practice the shift of the upper body to the bend's inside
- Pass over small obstacles with matching speed
- Avoid obstacles higher than 5 centimetre

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## Adjusting your Trets Reh

The proper adjustment of the Trets Reh is important for riding comfort and performance. The Trets Reh can be adjusted to fit riders between approx. 1.10 and 1.50 m (3'7"- 5') in height.

### Checking the seat position

Have the child sit in the seat of the Trets Reh and place its feet on the pedals with the balls of her feet over the middle of the pedal axles. When the crank arms are lined up with the front boom, one leg should be almost straight (see photo, left). If the leg cannot be straightened or the foot does not fully reach the pedal, then the length must be readjusted. The adjustment range is 26 cm (10 1/4").

### Adjusting the frame length

1. Loosen the Allen bolt between the front boom and the main frame (1) using a 6mm Allen key.
2. Push the front boom further into the main frame or pull the front boom further out of the frame.

#### Note:

If the front boom seems to jam before it reaches the point of complete insertion, check whether the tube of the quick-adjust frame assembly is hitting against the screw connection. In this case, do not try to force the front boom past this point.

#### Caution!

The front boom cannot be extended past the minimum insertion mark, which is a small hole located on the top of the boom. This hole should not be visible. If the hole is visible, the front boom has been extended past its minimum insertion length. Riding with an over-extended boom could cause accidents or damage to the frame.

3. Tighten the Allen bolt (1) lightly.
4. Check the seat position. If further adjustment is required, repeat steps 1 to 3.
5. Position the bottom bracket axle so that it is parallel to the surface of the ground.
6. Tighten the Allen bolt (1) between the front boom and the main frame using a 6mm Allen key and a torque of 12 Nm (8.9 ft-lbs).

#### Caution!

It is absolutely necessary to tighten these bolts with a torque of 12 Nm (8.9 ft-lbs). Riding with loose bolts poses a high risk of accident and damage to the frame.

7. Adjust the length of the chain tube to the frame length by sliding the inside tube further into (or out of) the outside tube.

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### Adjusting the handlebar height/angle

You can adjust the height of the handlebar by raising or lowering the handlebar stem. The handlebar angle can be adjusted by either changing the angle of the handlebar stem or tilting the handlebar itself.

#### Handlebar height

1. Loosen the bolt (1) using a 5mm Allen key.
2. Reposition the handlebar by sliding the handlebar stem in or out.

#### Caution!

Never ride the trike with the handlebar stem extended beyond the minimum insertion mark. Otherwise, the steering tube could break under pressure. Major risk of accident!

3. Tighten the bolt (1) using a 5mm Allen key.

#### Handlebar angle

4. Loosen the nut (2) with a 13mm open-end wrench. Brace the bolt using a 6mm Allen key. Adjust the position of the handlebar stem.
5. Retighten the nut (2).

#### Position of the grips

6. Loosen the two handlebar clamp bolts (3) in the handlebar stem using a 5mm Allen key.
7. Tilt the handlebar into the desired position.
8. Retighten the two handlebar clamp bolts using a 5mm Allen key.
9. Finally, loosen the two bolts of the brake levers using a 4mm Allen key and reposition the brake levers as appropriate.
10. Retighten the two bolts of the brake levers.

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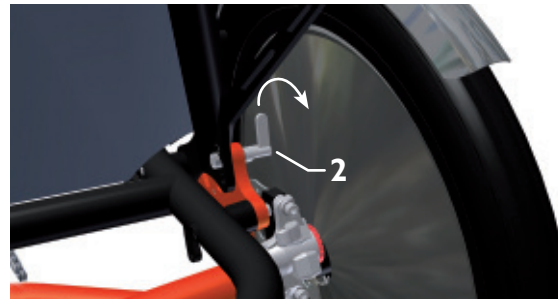
### Adjusting the brake levers to hand size

To position the brake lever closer to the handlebar grip, turn the adjusting bolt (1) clockwise using a 2mm Allen key. To position the brake lever further from the handlebar grip, turn the bolt counterclockwise.

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### Adjusting the seat angle

1. Release the two locking mechanisms (2) on the seat stays.
2. Adjust the seat angle to the desired position.
3. Attach the two locking mechanisms (2) so that they engage in one of the three holes in the seat stays.

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### Adjusting the safety belt

The safety belt can be adjusted to the dimensions of the rider (width and height). Adjust the safety belt so that the straps hold the rider snugly but are not uncomfortably tight.

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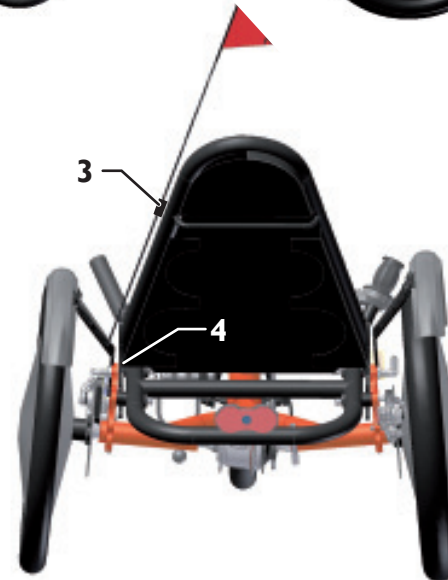
## Riding your Trets Reh

### Mounting and dismounting

The handlebar stem can be folded forward to allow for easier mounting and dismounting.

1. Gently pull the wire cable (1) between the stem and the headset towards you to disengage the locking mechanism (2).
2. Raise the handlebar stem to make additional space for mounting or dismounting.
3. After you have mounted or dismounted the trike, lower the handlebar stem and lock it back into place.

### Attaching the flag

#### Caution!

The Trets Reh is a relatively low vehicle. As a result, the trike is less visible to other road users. The Trets Reh comes with a safety flag that improves the trike's visibility to other road users. Be sure to ride with this flag at all times.

Guide the pole of the flag through the loop (3) on the left side of the seat cover (or on the right side when riding in countries with left-hand traffic), and secure the end by wedging it in the lower part of the seat stay (4).

### Correct tire pressure

The Trets Reh should only be ridden with the correct tire pressure. The tire pressure range is specified on the tire sidewalls. The two rear wheels are equipped with spoke guards. In order to reach the valve, pull apart the Velcro closures.

#### Caution!

Tire pressure affects the handling and safety of the Trets Reh. Therefore, always ride with the tire pressure indicated on the side of the tire.

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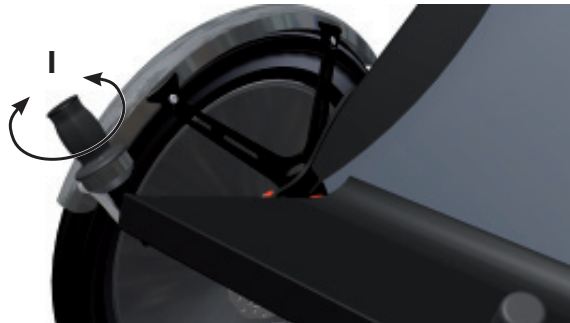
## Shifting

Shifting gears allows you to adjust pedaling strength to the terrain and desired speed. The force required per pedal revolution can be decreased by selecting the appropriate gear. In this way, it is much easier to ride up a steep hill in a lower gear than in a high gear. In a lower gear, however, you are slower and have to pedal faster. High gears are best suited for riding at higher speeds on flat terrain or riding downhill. Select a gear that allows you to pedal at more than 70 pedal revolutions per minute. This cadence is optimal for producing sustained power and preventing excessive strain on the knee joints. In its standard version, the Trets Reh is equipped with an 8-speed internal gear hub with twist shifter. The gear hub allows you to shift gears even when the trike is stationary.

### Caution!

Please note the information provided in the included manual from the gear system manufacturer.

### Caution!

Before riding the Trets Reh on public roads, the rider should familiarize herself with gear shifting in an open area away from traffic. Shifting gears takes practice, which could distract the rider from navigating road traffic.

## Shifting gears with the twist shifter

Turn the twist shifter (1) on the right grip clockwise to shift to a higher gear for faster riding or counterclockwise to shift to a lower gear for slower riding.

## Brakes

All standard model Trets Reh trikes are equipped with mechanical disc brakes. The brake levers (2) are mounted on the left and right grips of the handlebar. The left-wheel brake is controlled with the left brake lever, and the right-wheel brake, with the right brake lever.

### Caution!

The disc brakes of the Trets Reh are very powerful and have good modulation. However, it is still possible for the wheels to become locked. Before the child's first ride in traffic, be sure to have her test-ride the trike in an open area away from traffic.

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### Transporting luggage

Special mounts (1) for a bicycle bag are located behind the seat. These mounts are suitable for various models of bags. We recommend our Hase Luggage Bag. Other hook-mounted bicycle bags, such as Ortlieb or VauDe bags, can also be used.

### Fender-stay pouches

One or two additional pouches (optional) can be mounted to the fender stays (2) for transporting small items. These pouches have been designed for use with the HASE Kettwiesel trike; they are also compatible with the Trets Reh but cannot be filled to their full capacity due to the restricted space between the seat and the fender stays.

### Parking brake

The Trets Reh is equipped with a parking brake to prevent it from rolling when parked on a slope. Trikes with mechanical disc brakes have a lever (1) on the right handlebar grip.

1. Move the lever forward.
2. The brake is now engaged.
3. Move the lever back. The brake is disengaged.

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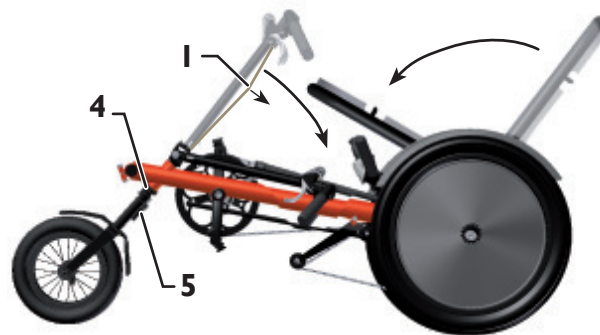
## Parking the Trets Reh upright

The Trets Reh can be parked upright with the front wheel elevated. In this position, it requires less space than a standard bicycle. To park the Trets Reh upright, simply lift the front wheel until the trike balances in this position.

## Transporting the Trets Reh

If you need to transport the Trets Reh by car, train, or other vehicle, you can make it more compact by reducing the frame length (see also page 9, “Adjusting the frame length”), folding the handlebar down and the seat forward, and removing the fork.

1. Mark (for example, with a permanent marker) the position of the front boom in the main frame on the bottom of the tube. This makes it easier to reassemble the trike with its original dimensions.
2. Adjust the frame to its minimal length. For more information, see the section “Adjusting the frame length” on page 9.
3. Pull the wire cable (1) between the stem and the headset gently towards the seat to disengage the locking mechanism.
4. Fold the handlebar down against the frame.
5. Release the two locking mechanisms (2) of the seat and fold the seat forward. Lock the seat in the transport position.
6. Open the quick release on the fork (4), press the “Snap Button” (5), and remove the fork with the front wheel.

### Caution!

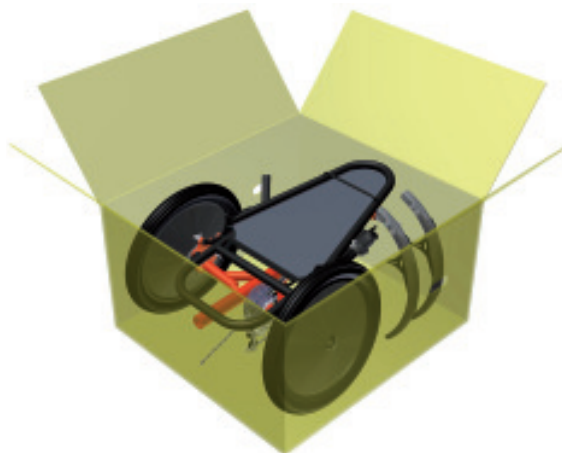
Be sure to protect all sensitive parts, such as the disc brakes and chain tensioner, from damage.

If you are planning to take the Trets Reh on an airplane or have it shipped by a forwarding company, we recommend packing it in a cardboard box for protection.

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### Packing the Trets Reh in a cardboard box

Use a cardboard box with internal measurements of at least 93 cm x 87 cm x 52 cm (37 x 35 x 21 in). Prepare the Trets Reh for transport as described above. Then place it in the box, and secure it with packing material to prevent movement. Be sure to protect all sensitive parts from damage.

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### Operating the lighting system

The Trets Reh comes with a front and rear light. The system is powered by a dynamo which runs sideways on the right rear wheel.

**Enable the dynamo** : Push the red button down.

**Disable the dynamo** : Pull the dynamo away from the tire.

The pushing force has to be just enough that the dynamo doesn't slip of the tire. This depends on the model of the tire and the weather. If the light is flickering, heighten the pushing force by turning the adjusting knob clockwise.

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### Maintenance intervals

**Note:**

Please have all inspection and maintenance work performed by your authorized Hase retailer. You may find an inspection pass at the end of this manual.

The first inspection should be done after the initial 300 km (200 miles). All additional inspections should be performed in 2000km (1,250-mile) intervals or on a yearly basis (see also page 50, “Inspection pass”).

Component	Action	Before every ride	Once a month	Once a year
Lighting	Check function	X		
Tires	Check tire pressure Check tread wear and sidewalls	X X		
Brakes	Check pad thickness Check alignment to the brake disc Test braking effectiveness	X	X X	
Brake cables	Check for damage	X		
Dynamo mounting	Check for tightness	X		
Chain	Check for wear Oil		X X	
Crank bracket	Check bolts			X
Wheels	Check for lateral/radial true Check spoke tension		X X	
Handlebar	Inspect			X
Headset	Check for play			X
Hubs	Check for play			X
Pedals	Check for axle play			X
Gear cables	Check for damage		X	
Quick releases	Check for tightness	X		
Bolts and nuts	Check for tightness		X	
Bottom bracket assembly	Check for play			X

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### Checking wheels for lateral/radial true

The wheels have been carefully trued prior to shipment. During the break-in period, stress on the wheels may cause the spokes to settle into the rims so that the spoke tension is no longer uniform and the wheels untrue. Have your wheels re-trued by your authorized retailer at the first inspection after approx. 300 km (200 miles). Subsequent tensioning is seldom necessary. You can check whether a rim is true by attaching a wire to the fork or fender stay, positioned so that it almost touches the rim. Then lift the Trets Reh, and rotate the wheel. If portions of the rim move away from or come in contact with the wire, the wheel must be taken to a qualified bicycle mechanic for re-truing.

#### Note:

The rear wheels do not have to be removed for the truing process. The spoke guards must be removed.

### Checking tires

Check the tires for damage and tread wear. Tires with cuts, cracks, or worn tread should be replaced. Check tire pressure before each ride and always ride with the pressure specified on the tire sidewalls.

#### Caution!

Never ride with excessive tire pressure (higher than specified on the sidewall). The tire could burst. Risk of accident and injury!

### Changing tires or repairing a flat

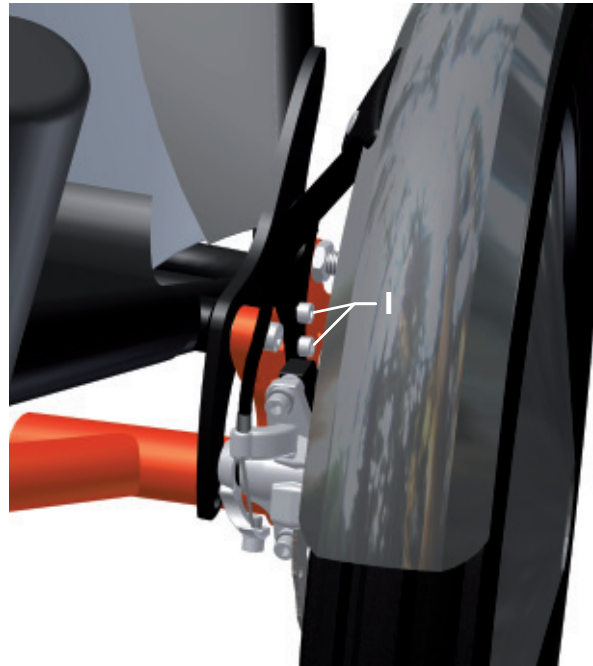
The front wheel of the Trets Reh is mounted with a quick release. Simply release it to remove the front wheel. You do not have to remove the back wheels for changing a tire or repairing a flat. The tires can be removed and re-mounted using a standard tire lever with the wheels attached. When removing and remounting tires, take care not to damage the fenders, and check that the rim tape is positioned correctly on the rim. The spoke guards must be clamped between the tire and the rim.

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**Note:**

If you replace the original tire with a tire of a different size, it may be necessary to readjust the fenders and the position of the dynamo.

**Adjusting the fenders**

1. Loosen the two Allen bolts (1) using a 5mm Allen key.
2. Slide the fender stay in the slot until it reaches the desired position, ensuring that the fender does not rub against the tire.
3. Tighten the two bolts (1) using a 5mm Allen key.

**Checking disc brakes**

Please note the information provided in the enclosed manual from the brake manufacturer. Have a qualified bicycle mechanic check the pad thickness regularly. If you notice any unusual braking sounds or inadequate braking strength, consult your bicycle mechanic immediately.

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### Checking the chain

The chain of the Trets Reh is longer than that of a standard upright bicycle. A long chain is generally subject to less wear. Therefore, it does not have to be changed as often as shorter chains. You can check the condition of the chain using a chain wear gauge from a bike shop. We recommend that you have a qualified bicycle mechanic check the condition of the chain and, if necessary, replace the chain, and possibly also the chainring and sprocket.

The chain required for the longest frame adjustment of a Trets Reh is 2336.8 mm (92 in) in length. This is equivalent to 92 chain links.

### Cleaning and oiling the chain

In order to ensure that the chain does not wear out before the end of its maximum working life, it must be cleaned and oiled on a regular basis.

1. Clean the chain using a clean, dry cloth.

#### Caution!

Never use solvents or pressure cleaners. These could wash the lubricants out of the chain bearings and shorten the life of the chain.

2. Apply chain oil, grease, or wax evenly to the chain links while slowly rotating the crank.

#### Note:

Rain can wash off traces of lubricant, releasing it into the environment. We recommend biodegradable lubricants.

#### Caution!

Never allow chain lubricant to come in contact with the brake discs or tires. This could significantly reduce braking performance or, in the case of lubricant on tires, wheel traction.

3. Continue rotating the crank to distribute the lubricant.
4. Wait several minutes to allow the lubricant to penetrate. Then wipe off all excess lubricant using a clean, dry cloth.

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### Checking the chain tube

The purpose of the chain tube (1) is to protect the clothing from chain lubricant. Over time, friction from the moving chain will cause the tube to wear. Check the ends of the chain tube for any signs of wear. If the ends are completely worn, have your bicycle mechanic replace the chain tube.

#### Note:

In order to replace the chain tube, the chain must be opened at the connecting link.

### Cleaning and disinfecting your Trets Reh

Clean your Trets Reh regularly. Dirt, as well as salt from winter roads, can damage the trike. Clean the trike using a sponge and soapy water (ideally with a biodegradable household cleaning product). If there are any persistent stains on matt coating, they can easily be handled with rim-cleaner available at car specialized trades.

You can also clean your trike using a gentle stream of water. However, do not use a pressure washer. The pressurized water can penetrate bearings and damage them. After cleaning the trike, clean and lubricate the chain. Wipe all surfaces with a moist, lint-free cloth.

The handlebar grips and seat can be cleaned and disinfected using household cleaning agents and disinfectants. Only use disinfectants that do not damage plastic parts, e.g. aldehyde-based disinfectants.

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## Appendix for dealers

### Technical data

Total length: 128-158 cm (50½-62¼ in)
Minimum length without front wheel: 100 cm (39½ in)
Total width: 75 cm (30 in)
Total height: 87 cm (34¼ in), (With seat folded: 52 cm / 20½ in)
Weight: 17.5 kg (38.5 lbs)
Permissible load: max. 65 kg / 143 lbs (If the Trets Reh is used as a trailer, the max. permissible load is 45 kg / 99 lbs. Be sure to observe the applicable laws for bicycle trailers in the country of use. In Germany, the legal weight limit is 45 kg (99 lbs) for trailers without overrun brakes.)
Brake system: mechanical disc brakes with parking brake
Gears: 8-speed internal gear hub
Frame: aluminum, powder-coated
Rear wheels: 20", front wheel: 12"
Rims: aluminum
Tires: Rear 35-406 to 60-406 Front: 50-203 Tire-pressure range: use recommended tire pressure shown on sidewalls.
Fits riders from 1.10 to 1.50 m (3'7" to 5') in height

### Tightening torques

Use a torque wrench for all bolts. The following table lists the standard values for bolts of 8.8 quality (Grade 5)

Bolt	Tightening torque
M 5	4 ft-lbs
M 6	6.6 ft-lbs
M 8	17 ft-lbs
M 10	34 ft-lbs
M 12	58 ft-lbs

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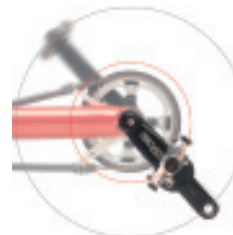
**Technical data of the accessories  
(see page 36)**

**Material for one-handed handling**

Weight	72 g
Material	Kunststoff, Aluminium, Edelstahl
Dimensions Tube length, ø Bracket length Bracket height	66 mm, 22 mm 63 mm 16 mm

**Walker-Mount**

Weight	98 g
Material	Kunststoff, Edelstahl
Dimensions Cup ø Height	50 mm 64 mm
Dimensions mount Height Width Length	39 mm 66 mm 74 mm

**Crank shortener with pedal pendulum**

Weight	348 g
Material	Aluminium, Edelstahl
Dimensions pendulum Length Width Height	120 mm 40 mm 15 mm
Dimensions crank shortener Height, ø Max. crank-width to mount	23 mm, 60 mm 36,5 mm

**Crank shortener**

Weight	154 g
Material	Aluminium, Edelstahl
Dimensions	s.o.

**Special pedals with/without calf support**

Weight calf support Weight special pedals	310 g 580 g
Material	Aluminium, Edelstahl, Kunststoff
Dimensions calf support Length Width Height	340 mm 110 mm 35 mm
Dimensions special pedals Length Max. height Min. height	175 mm 200 mm 130 mm

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**Blinker system**

Weight	360 hg
Material	plastic, aluminium
Dimensions Tube length, ø	350 mm, 12 mm
Dimensions of single blinker	
Length	65 mm
Width	55 mm
Height	50 mm

**One-sided leg rest**

Weight	718 g
Material	plastic, aluminium
Dimensions	
Length of foot rest	275 mm
Length of calf rest, ø	105 mm, 22 mm
Length total	500 mm
Width total	180 mm

**Fixed gear (with sliding hitch)**

Weight	468 g
Material	aluminium, stainless steel
Dimensions	
Height, ø	108 mm, 80 mm

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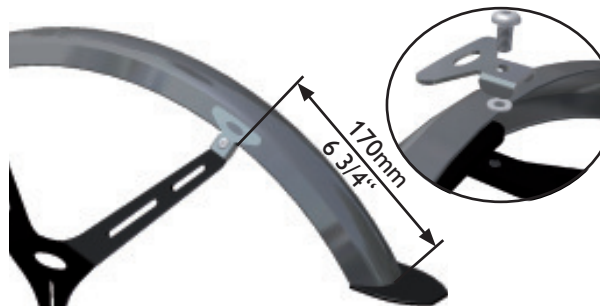
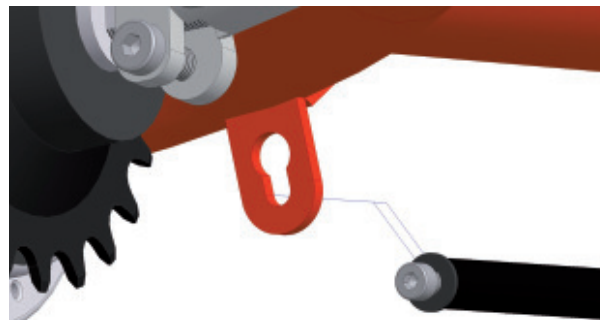
**Accepting delivery**

If you notice the packaging has been damaged, please notify the shipping company directly.

**Unpacking and checking contents**

Open the packaging and remove the shipping supports. Check the contents of the package for agreement with your order. In addition to the Trets Reh, the following items should be included: flag, the Trets Reh User's Manual, and manuals from other manufacturers for various components, such as gear system, brakes, lighting system, etc.

**Assembling the Trets Reh**

1. Lift the seat and lock it into place.
2. Carefully pull out the front boom and mount the quick-adjust frame assembly by inserting the bolt at the end of the tube into the keyhole-shaped slot on the frame under the seat and tightening the bolt.
3. Adjust the frame length (see page 9).
4. Mount the fork with the front wheel.
5. Mount and align the handlebar.
6. Attach the fenders to the fender stays.
7. Mount any other accessories that were ordered.

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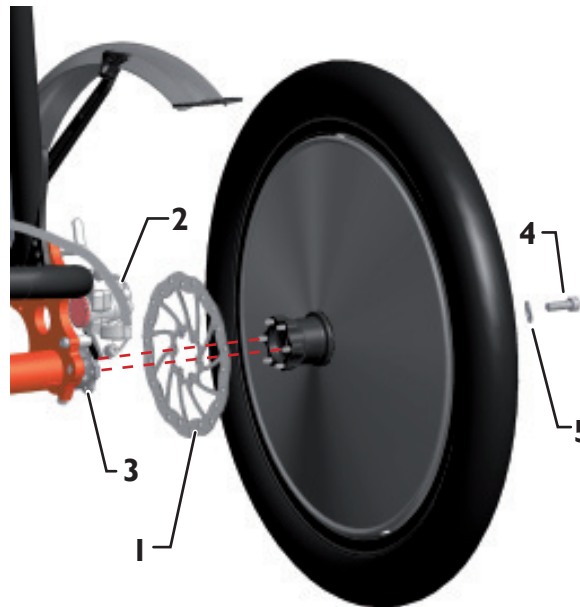
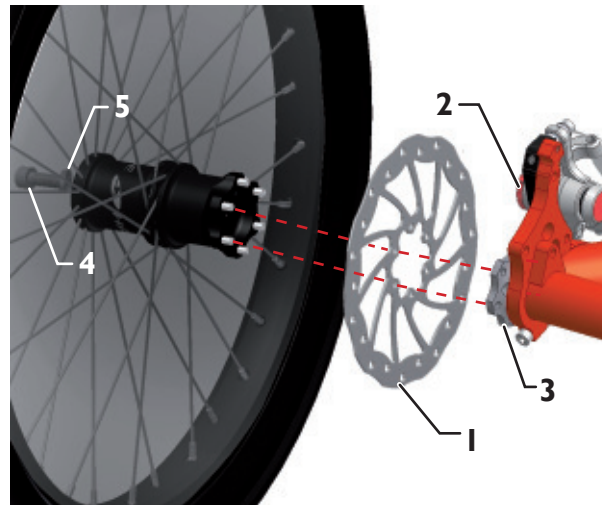
### Removing the rear wheels

For easier wheel removal, position the trike so that the wheels can rotate freely and are not in contact with the ground. For this purpose, the Trets or Trets Trike can be mounted to a work stand. If no stand is available, you can also raise the Trets or Trets Trike off the ground by placing something under the frame. If the Trets is equipped with derailleur gears, only the right wheel is driven. If the Trets has an internal gear hub, only the left wheel is driven.

1. Brace the wheel.
2. Unscrew the M8 bolt (1) by turning it counterclockwise using a 6mm Allen key.
3. Remove the wheel while holding the brake disc to keep it from falling off.

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### Mounting the rear wheels

1. Slide the brake disc (1) between the two brake calipers (2) from below, and hold it against the head of the drive screw (3).
2. Hold the left wheel against the brake disc (1) so that the hub pins protrude through the holes of the brake disc (1) and into the holes in the head of the drive screw (3).
3. Hold the Nord-Lock locking washer (5) against the hub, and screw the wheel onto the frame with the M8 bolt (4) using a 6mm Allen key, tightening lightly at first.
4. Brace the wheel, and tighten the M8 bolt (4) by turning it clockwise with a torque of 23 Nm (17 ft-lbs).

#### Caution!

The M8 bolts that fasten the wheels to the hubs must be used with Nord-Lock locking washers and tightened with a torque of 23 Nm (17 ft-lbs). Otherwise, the fasteners could loosen over time and cause accidents while riding.

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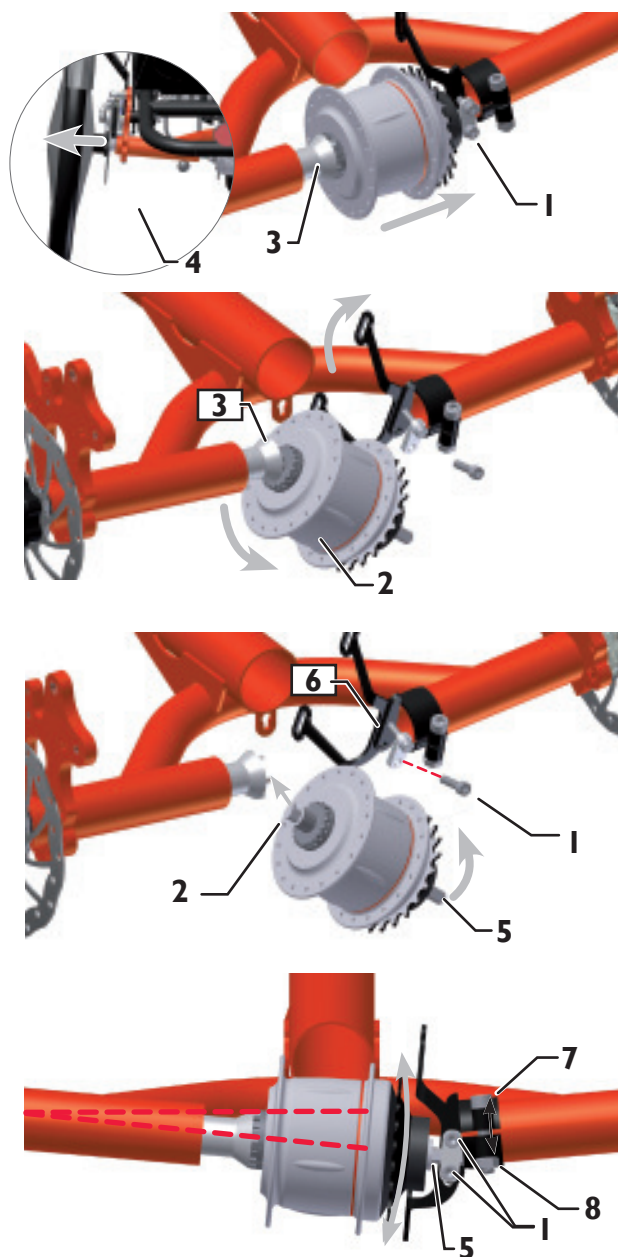
### Removing the internal gear hub

1. Lift the chain, pull it off of the sprocket, and lay it over the right axle tube.
2. Remove the lower bolt (1) of the torque support and loosen the upper bolt. Push the hub to the right to make space between the hub and the gear hub connection (3).
3. Loosen the M8 bolt (8) and rotate the torque support and the chain guard holder to the back.
4. Pull the left axle of the hub (2) out of the gear hub connection (3).

#### Note:

If you are not able to pull the left axle of the hub (2) out of the gear hub connection (3), you must remove the left rear wheel and brake disc. Then loosen the bolt (4) of the left wheel components, and push the wheel components a few millimeters to the left.

5. If necessary, remove the gear cable.

### Installing and adjusting the internal gear hub

Requirements: fully installed left wheel components with gear hub connection, right wheel components, and torque support.

1. Pull the chain around the right axle tube.
2. Loosen one or both bolts (1) of the torque support if they are tightened down.
3. Fit the left axle of the hub (2) with the external tooting into the tooting of the gear hub connection (3).
4. Tip the right axle of the hub (5) upwards into the mount of the torque support (6).
5. Position the gear hub so that the external tooting of the hub fits as snugly as possible into the tooting of the gear hub connection. In this position, the hub will not be completely horizontal but rather aligned with the left axle tube. This positioning can be achieved by sliding the right axle of the hub (5) into the mount of the torque support (6) and, if necessary, rotating the torque support (7).
6. Tighten all the bolts (1) and (8) of the torque support.
7. Pull the chain onto the sprocket.
8. Remount the gear cable (if applicable).

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
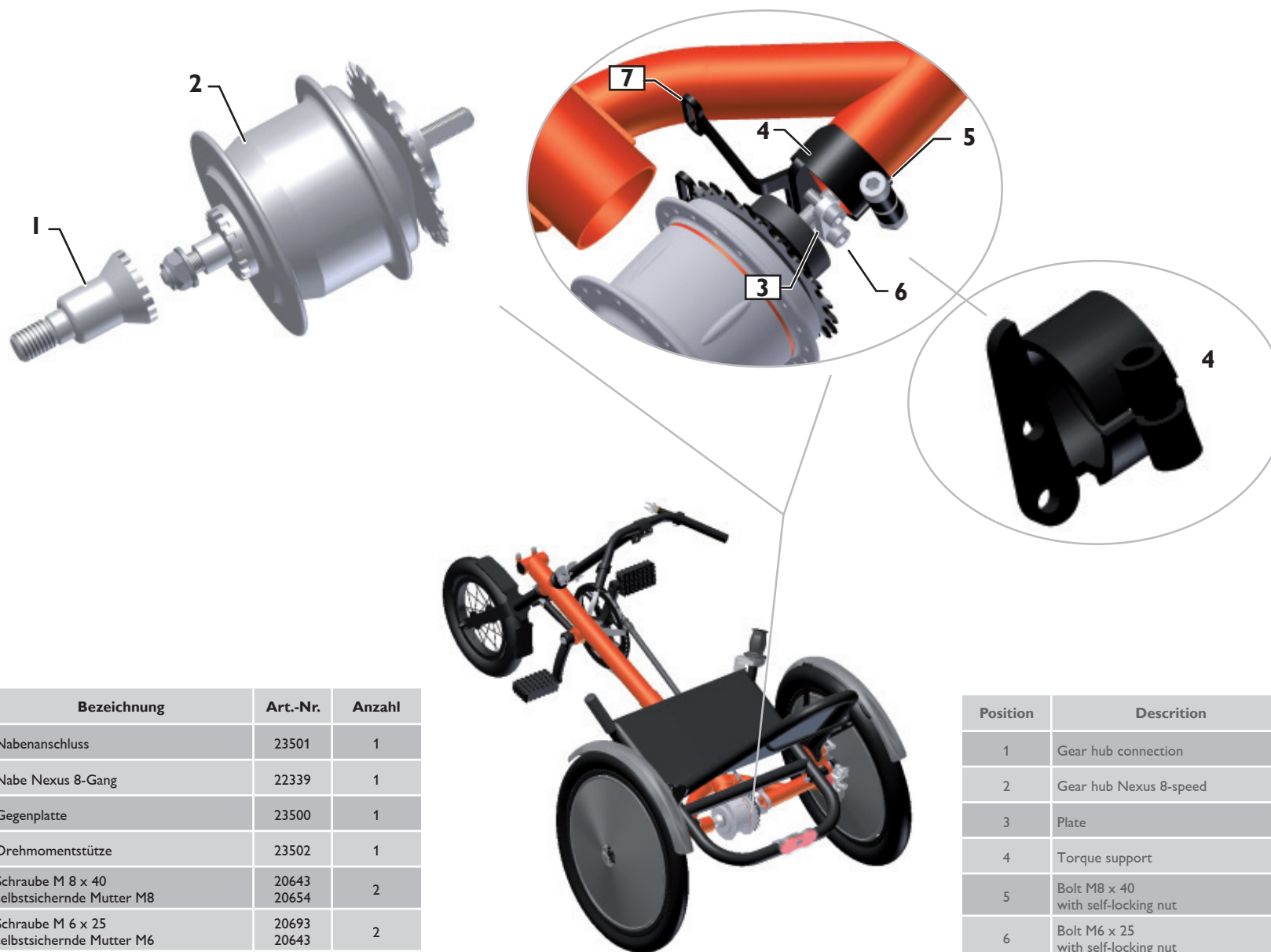
Position	Description	Art.-No.	Qty
1	Gear hub connection	23501	1
2	Gear hub Nexus 8-speed	22339	1
3	Plate	23500	1
4	Torque support	23502	1
5	Bolt M8 x 40 with self-locking nut	20643 20654	1
6	Bolt M6 x 25 with self-locking nut	20693 20643	2
7	Chain guard holer, Trets, rear	24773	1

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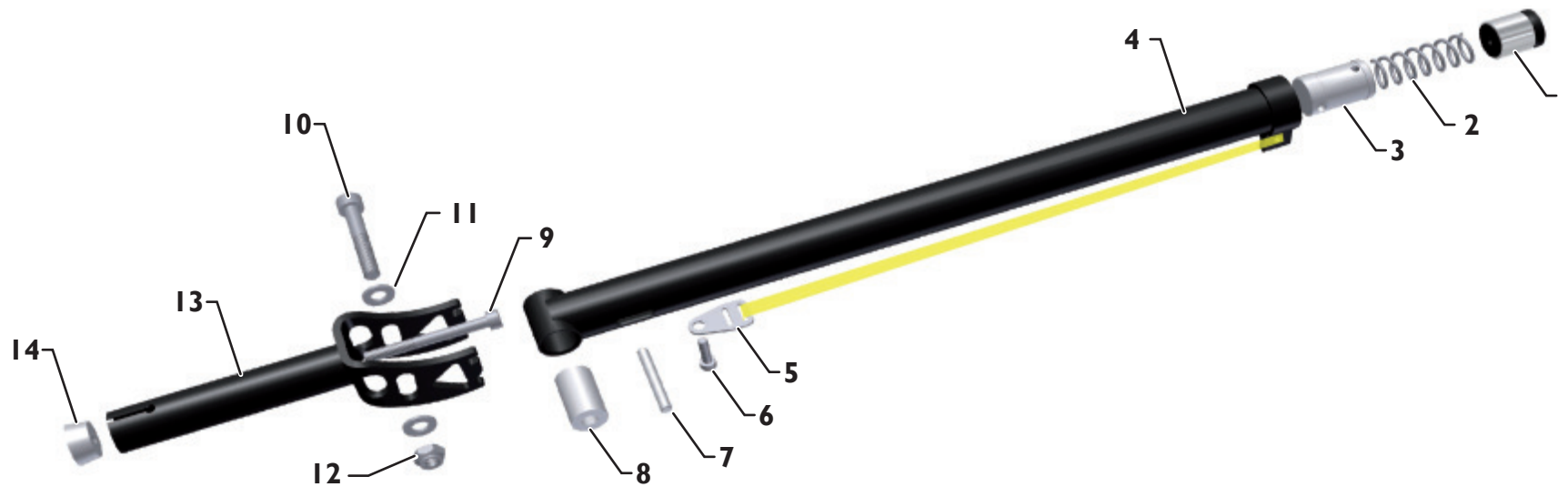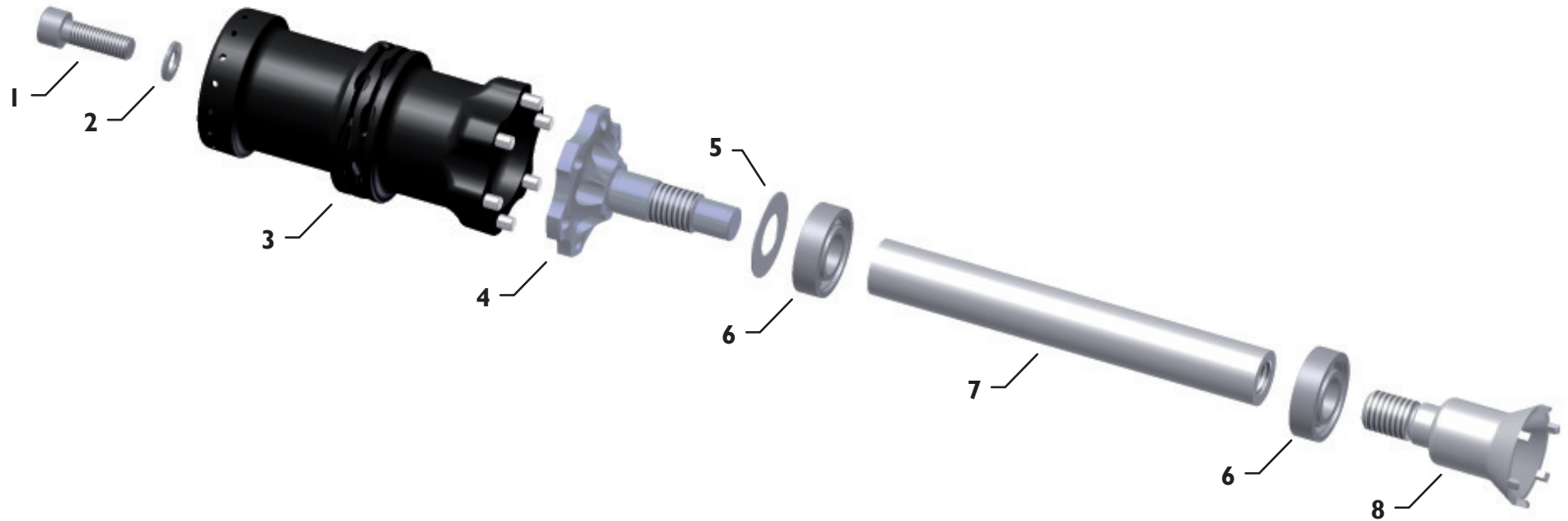
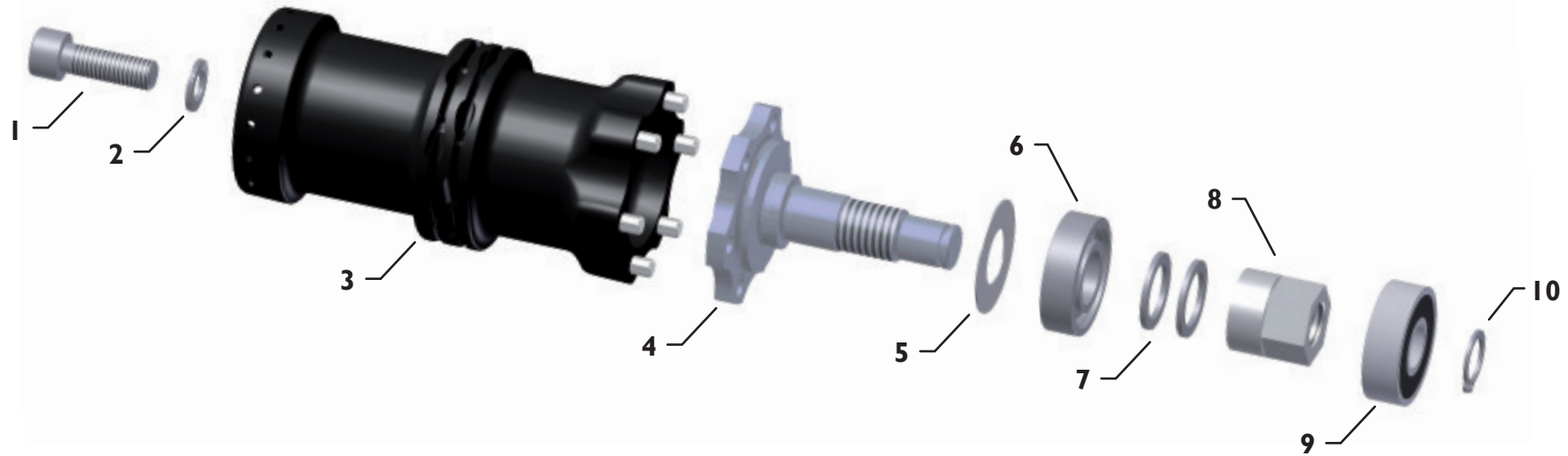
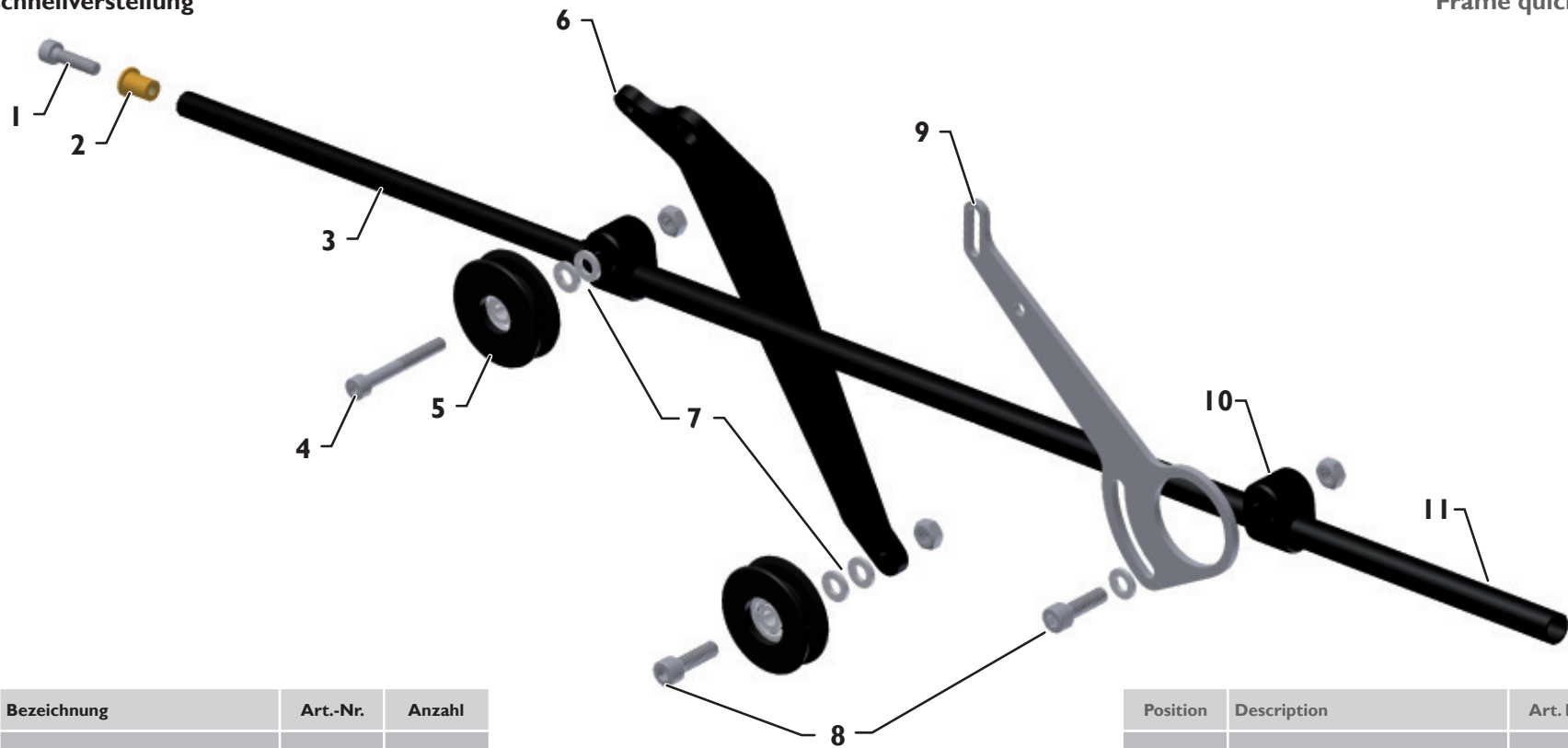
For people who use a walking aid.

### Features

One hand is all the rider needs to place the walking stick in the cup (1) and fasten it using the stick holder (2).

### Assembly and Adjustment

The holders could be mounted to tubes with 15 bis 60 mm in diameter.

1. Place the strap (3) around the tube, lay the end of the strap through the slot of the tension plate (4) and band it. Cut it 10 mm after the tension plate (4).
2. Put the socket (5) on the tension plate (4).
3. Press the clamp plate (6) through the hole in the cup (1) or the stick holder (2).
4. Put the cup (1) or the stick holder (2) with the clamp plate (6) to the socket (5) and tighten the bolt (7) lightly.
5. Adjust the cup (1) or the stick holder (2) and tighten the bolt (9).

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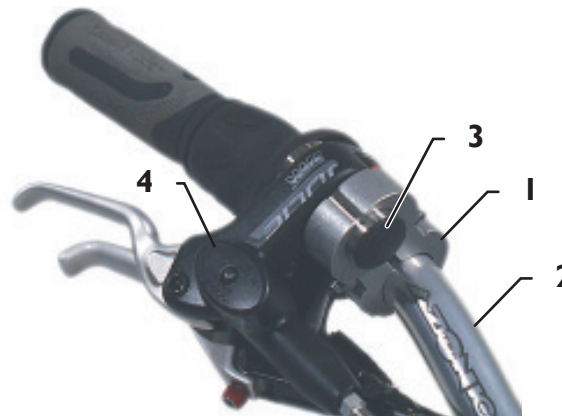
## One-Hand Controls

### Intended Application

For stroke patients or hemiplegics.

### Features

Two independent braking systems with separate brake levers are mounted to a single handlebar end, making it possible to steer and brake using a single hand. German law requires two independent braking systems.

### Assembly and Adjustment

An additional handlebar segment (3) with the second brake lever (4) is mounted to the handlebar (2) with a clamp (1).

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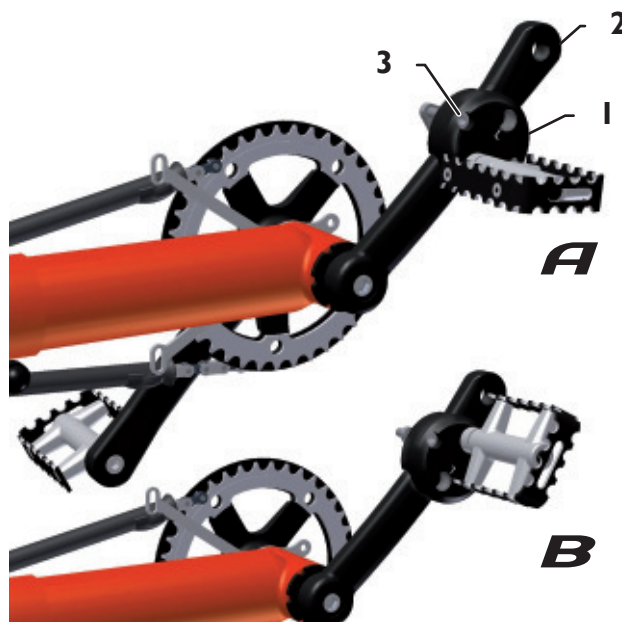
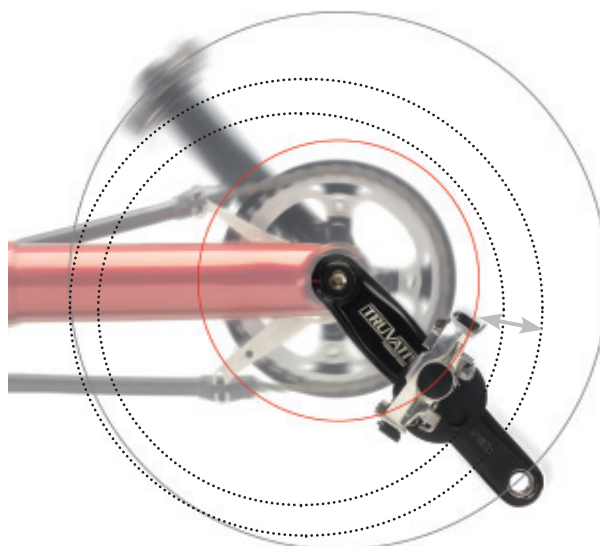
## Crank Shortener

### Intended Application

Designed for people with restricted leg mobility or with legs of different lengths. Can also be used for adjusting the crank to fit children.

### Features

The crank shortener reduces the pedal radius. The pedal radius is adjustable, allowing the rider to train leg mobility and increase pedal radius gradually as mobility increases.

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The crank shortener has two threaded holes for the pedals, making it possible to mount the shortener on either side. Please pay attention to the markings: "L" stands for left-hand thread and is intended for the left pedal. The threaded hole with no marking is intended for use with the right pedal.

**Please use the appropriate hole. Mounting errors can damage the crank shortener!**

2. Screw the pedal into the appropriate hole ("L": left-hand thread) of the crank shortener.

#### Caution!

If the end of the threaded pedal stem is protruding from the crank shortener after the pedal has been screwed on, then reattach the pedal, using a washer between the pedal and the crank shortener. Otherwise, the crank shortener cannot be attached securely enough, and the crank could become damaged.

3. Place the crank shortener (1) over the crank arm (2) and tighten the two Allen screws (3) lightly.

#### Note:

If you intend to shorten the crank significantly, then mount the crank shortener with the pedal inwards (closer to the axle) (see A). If you only wish to shorten the crank by a small amount, mount the shortener with the pedal outwards (further from the axle) (see B).

#### Caution!

The crank shortener cannot be mounted to curved, tapering, or plastic-coated cranks. It also cannot be mounted in the curved section of the crank. The two Allen screws must extend at least 5 mm into the rear section

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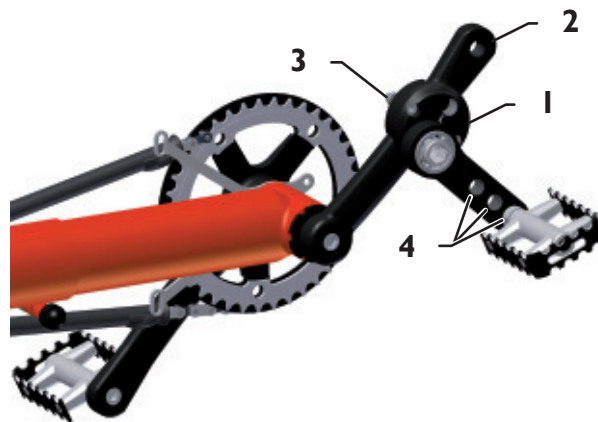
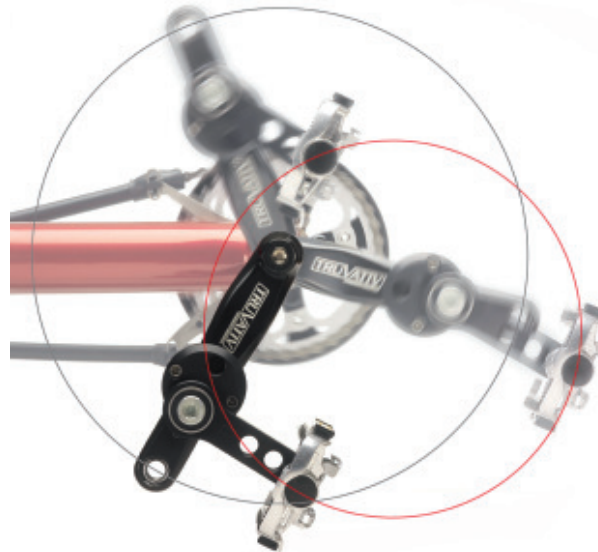
of the shortener. If this is not possible, replace the screws that came with the shortener by longer screws with a strength rating of 8.8.

4. Slide the crank shortener (1) along the crank arm (2) to find the optimal position. Then tighten the two Allen screws (3) securely.

## Pedal Pendulum

### Intended Application

Designed for people with limited flexibility in one leg.

### Features

The pedal pendulum reduces the pedal radius while maintaining the lower position of the pedal. The pedal radius is adjustable. This allows the rider to train leg mobility and gradually increase the pedal radius as mobility increases.

### Assembly and Adjustment

1. Unscrew the original pedal from the crank arm.

Please pay attention to the markings: “L” stands for left-hand thread and is intended for the left crank. The pedal pendulum with marking “R” is intended for use with the right crank.

**Please be sure to use the correct pedal pendulum. Mounting errors can cause damage to the pedal pendulum!**

2. Slide the pedal pendulum (1) onto the crank arm (2) and tighten the two Allen bolts (3) lightly.

#### Caution!

The pedal pendulum cannot be mounted to curved, tapering, or plastic-coated cranks. It also cannot be mounted in the curved section of the crank. Please ensure that the pedal pendulum does not collide with the ground or other bicycle parts when pedaling.

3. Screw the pedal (“L”: left-hand thread, “R”: right-hand thread) into one of the three holes in the pedal pendulum. Use of the outer hole will produce a larger pedal radius, and use of the inner hole will produce a smaller pedal radius.

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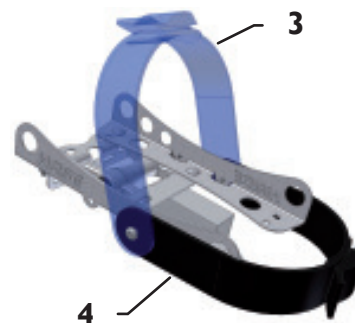
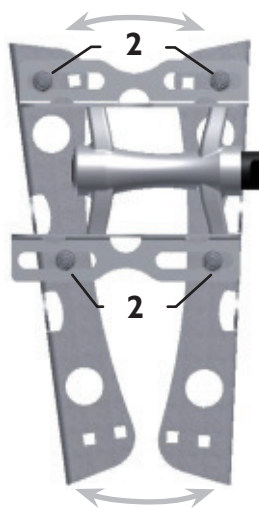
## Special Pedals

### Intended Application

These pedals have been designed for riders with mal-alignment or motor disorders of the feet.

### Features

The Special Pedals are screwed into the crank arms, replacing the standard pedals. Each pedal has four M5 clamp screws located inside special adjustment slots on its base. Each of these slots offers an adjustment range of 20 mm (3/4"). The foot can be slid forward or backward in the pedal, as required, and rotated by a max. angle of 35°. This makes it possible to adjust the pedal to the type and size of shoe and the position of the foot. Easy to use: The strap is unbuckled. The foot is placed on the pedal, slid back and wedged into the tapered side braces, and secured with a Velcro strap. If necessary, the holes in the pedal can be used for additional straps. A smaller model for children is also available.

### Mounting and Dismounting

1. Unbuckle the strap. Place the foot on the Special Pedal and slide it backwards until the shoe is wedged between the two side braces.
2. Tighten the strap(s).
3. For dismounting unbuckle the strap and slide the foot out from the braces.

### Assembly

The Special Pedals are screwed into the crank arms (1) in place of the standard pedals.

### Adjustment

We recommend performing these adjustments after the pedals have been mounted.

1. Loosen the four clamp screws (2).
2. Place the foot (of the rider) on the pedal so that the ball of the foot is aligned with the axle of the pedal.
3. Pull the side braces tightly against the sides of the shoe, and tighten the four clamp screws (2) using an 8mm open-end wrench.

#### Note:

If the pedal has been properly adjusted but the foot still slips out, a second strap (4) can be mounted in addition to the existing strap (3)

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## Special Pedals with Calf Support

### Intended Application

These pedals have been designed for riders with spasticity, peroneal paralysis (drop foot), or paralysis of the foot and calf.

### Features

Functions like the Special Pedals but with additional lower-leg support. The calf brace can be adjusted in height, proximity to the frame, and foot length. These pedals do not change the original orientation of the foot: i.e. the distance between the foot and the center of the frame is maintained.

### Mounting and Dismounting

1. Unbuckle the strap. Place the foot on the Special Pedal and slide it backwards until the shoe is wedged between the two side braces.
2. Tighten the strap(s).
3. Tighten the velcro strap(s) of the Calf Support.

### Assembly and Adjustment

Same Assembly Procedure as Special Pedals

1. Loosen the clamp screw indicated in the photo as (1). Now you can adjust the calf brace to the size of the foot.
2. Loosen the screw indicated in the photo as (2). You can now adjust the height and angle of the calf brace.

#### Caution!

These adjustments should only be performed by a trained professional. Incorrect adjustments could lead to serious medical complications or injury.

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When minor foot support is needed.

### Features

This is the simplest and most universal solution for securing the foot in the pedal. The straps can be individually adjusted to the size of the foot. The time required for assembly is minimal.

### Adjustment and Use

The toes are inserted into the toe clips (1) and the elastic band (2) is pulled around the heel. Adjust the length of the toe-clip strap and the elastic band to fit the foot. The length of the elastic band can be adjusted by changing the position of the knot in the band.

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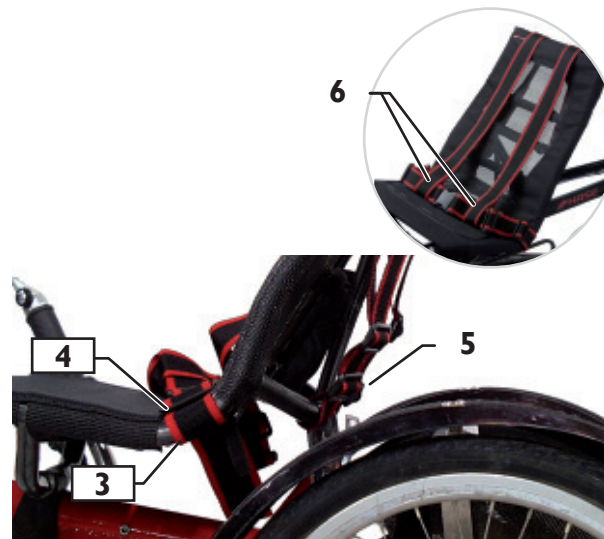
## Lap and Shoulder Belts

### Intended Application

For all riders whose physical condition puts them in danger of slipping or falling out of the seat when riding or braking.

### Features

The Lap Belt (1) only secures the lower torso. Full mobility for the upper torso remains. The Shoulder Belt (2) secures both the upper and lower torsos.

### Assembly and Adjustment

1. Wrap the Lap Belt (1) around the seat frame (3) and secure it using the plastic buckles (4).
2. Wrap the ends of the Shoulder Belt around the back seat tube or the rear bracing of the seat cover, and secure it using the plastic buckles (5).
3. Open the clasp of the Lap Belt and slide the loops (6) of the Shoulder Belt over the ends of the Lap Belt.

The belt can be loosened or tightened by adjusting the length with the plastic buckles.

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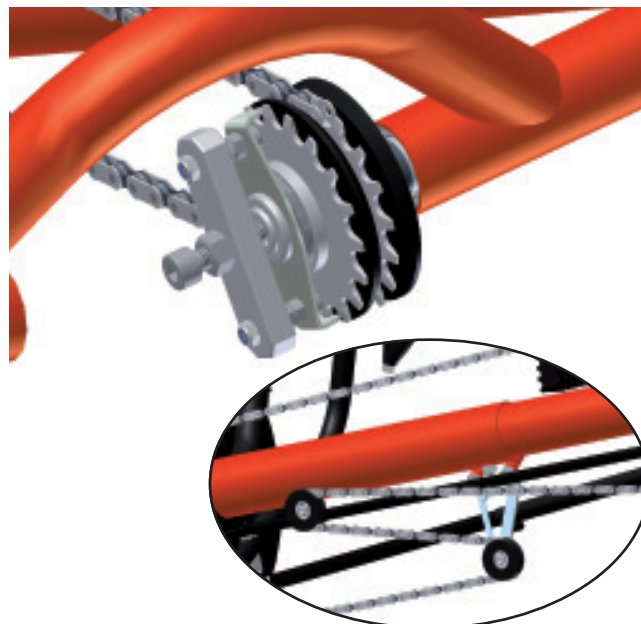
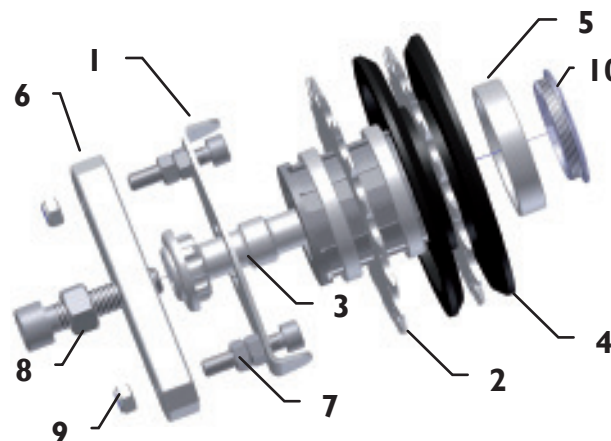
**Umlenkrollen demontieren**  
remove idler pulleys

### Fixed gear

The fixed gear allows you to get over the dead center at your Kettwiesel or Trets Trike with singlespeed. The chain takes the pedals and the slipping clutch allows you to hold the pedal in position by too fast revolutions. Can be switched on/off in seconds.

**Note:**

The fixed gear is designed only for Kettwiesel and Trets Trike without gearshift, except Speeddrive and Mountaindrive. The fixed gear cannot be used with the frame quick-adjustment. Remove the idler pulleys and shorten the chain adequate.

### Installing the fixed gear

1. Brace the wheel and unscrew the sprocket bolt (3) clockwise (attention: left-hand thread).
2. Put the singlespeed 18Z sprocket (2), protection caps (4) and the bushes (5) onto the freewheel according to the picture. Put the spring steel sheet onto the sprocket bolt.
3. Mount the drive. Use mid strength loctite and tighten the sprocket bolt with 79 Nm counter-clockwise.
4. Put the spring steel sheet (1) onto the M5 screws (7) and counter them with two M5 nuts on each side. Fit the support plate on it and tighten the self-locking M5 nuts (9). Adjust the slipping clutch with the M8 bolt (8). Tighten the M8 counter-nut.
5. To switch the freewheel on, loose the counternut and turn the M8 bolt until the spring steel sheet do not lock the singlespeed sprocket anymore. Tighten the counternut.

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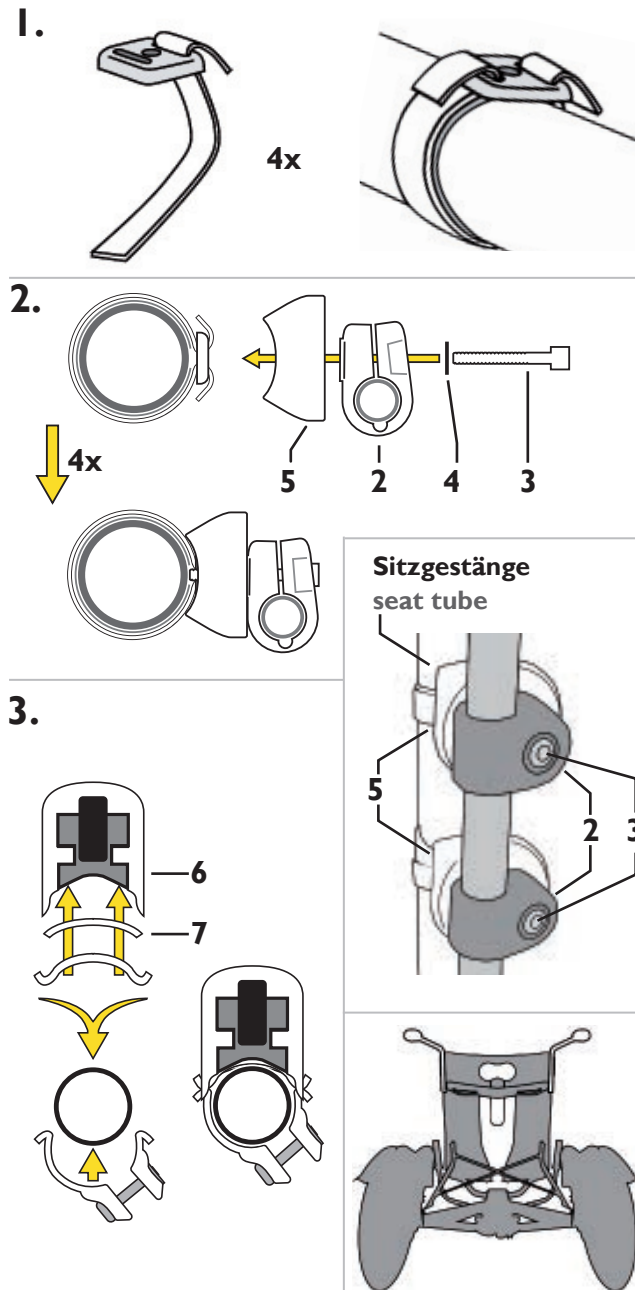
**Direction indicator lights LED**

The direction indicator lights LED can be updated on HASE-bikes. It is operated with a switch at the handlebar and runs with 4 Micro LR03,AAA cells.

**Assembly**

1. Loosen the seat cover. Insert one end of the stainless steel strip through the clamping piece and bend it around. Place the metal strap, with rubber pad underneath, around the seat tube and through the second slot in the clamping piece and bend it (fig. 1). Attach the second stainless steel collar (in about a distance of 10cm) to the seat tube.
2. Slide the plastic clamps(2) on the holding tubes of the indicators. Make sure that the side with the countersink shows to you. Run the M4x30 allen screw(3) with washer(4) through the plastic clamp(2) and the mounting part(5). Screw them all in the clamping piece slightly. Align the indicator and tighten the screws. Thereby the metal straps will tighten. Route the wires along the light cable to the handlebar.
3. Assemble the switch mount as shown in fig. 3, then fit the switch(6) on the handlebar and adjust it in a comfortable way so it can be operated easily during the ride.

**Attention:**

Ensure that the included isolation(7) (rubber sheet) is placed between the switch and the handlebar. Otherwise, it may result in a current flow between the switch contacts.

4. Fix the cables with cable straps to appropriate points (frame clamp / pivot of the seat). Notice not to affect other functions like the folding mechanism of the seat.

**Replacement of the batteries**

Open the light cases, on the left side of driving direction. Replace the 4 batteries on both sides. Close the light cases again by simply pushing the case parts together.

Nr	Description	Art.-No.	Qty
1	R&K Mount set	23487	4
2	Clamps PA	20921	4
3	M4x 30 Allen head bolt VA	20926	4
4	M4 Washer	20927	4
5	Mounting part		4
6	Switch	23247	1
7	Direction indicator lights LED	23980	1

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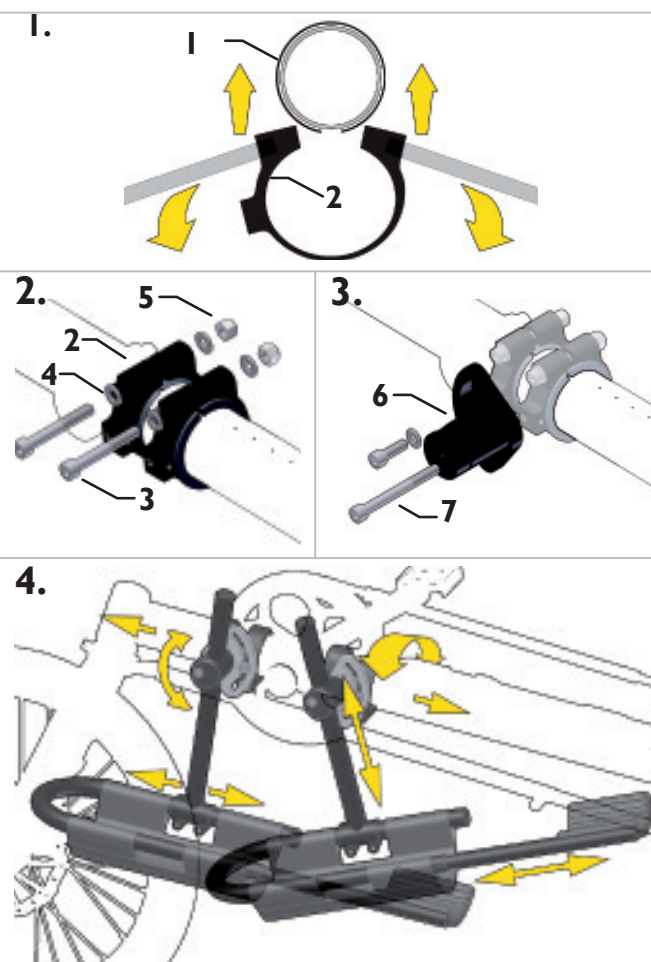
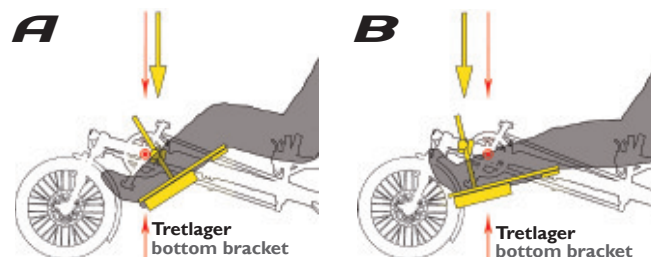
## One-sided leg rest

The leg rest fits our model KETTWIESEL as well as the LEPUS and can be used for both sides equally.

The assembly includes the pre-assembled leg-rest and the fitting mount. Furthermore four adapter pieces for mounting on the LEPUS, which are used instead of the slide bearings. Previous to the assembly you have to make sure which kind of permanent leg position you want to achieve (tilted or straight leg)

### For attachment you will need following tools:

- allen key (size 4,6,8), wrench (size 13), puller for crank handle
- hacksaw and metal-file (for assembly on the right)
- chain lock + riveter (for assembly on the right with tilted leg position)

### Assembly for left side KETTWIESEL

**A: rear bottom bracket (tilted leg position)**  
**B: ahead of bottom bracket (straight leg position)**

1. Remove the left crank handle with the allen key and the puller.
2. Disassemble the mount of the leg rest. Place the slide bearings(1) by bending around the main pipe behind/ in front of the bottom bracket. Carefully repeat this process with the joint plates(2) (via allen keys or comparable tools, fig. 1). Mind that the grip-spaces should face upward and the lateral openings to the preferred side for the rest. Push the slide bearings under the joint plates and hand-tighten them (with aid of grippers) by using one allen bolt(3), two washers(4) and a lock nut(5) for each joint plate(2). (fig. 2)
3. Mount the grip-cylinder(6) as shown in picture 3. **Tipp: To give additional adjustability, the bended oval hole can be attached facing forward or backward.** Hand-tighten all bolts except bolt(7) (leave it loose).
4. Make sure that the leg rest's assembly is fitting for your desired side (picture 4: left) . If applicable change the assembly. Slide the bar of the leg-rest into the grip-cylinder. Now hand-tighten bolt(7) too.

### Adjustment

**Picture 4:** First adjust the length of the KETTWIESEL's front end to fit the drivers proportions. Then set up the different hinge-angles and distances of the leg-rest. Afterwards thighten all nuts and bolts.(M6: 9Nm, M8: 23 Nm)

**Attention:** Please note that there must not be any touching point between the rest and the fender or the steering rod when on full lock. We recommend a minimal space of 3 cm to the fender.

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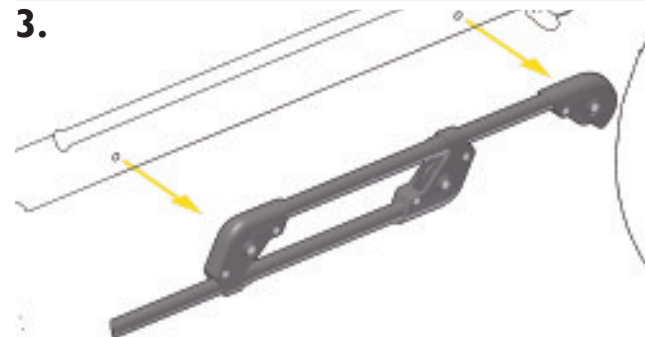
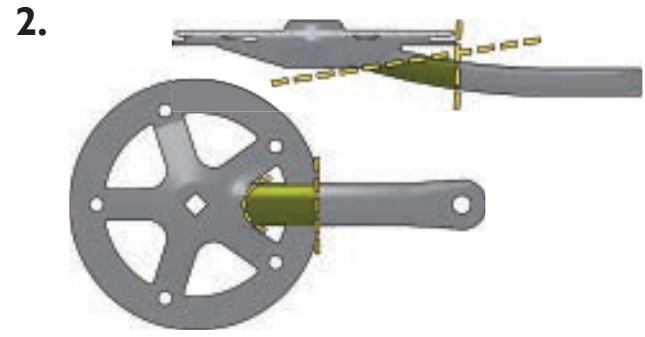
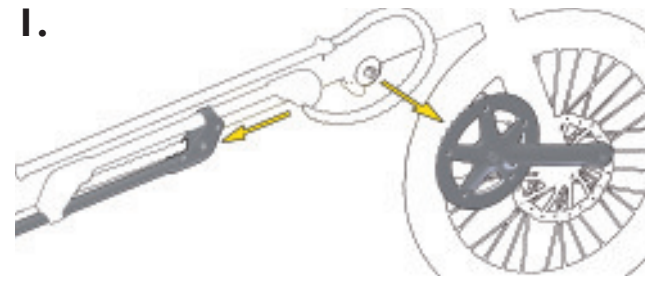
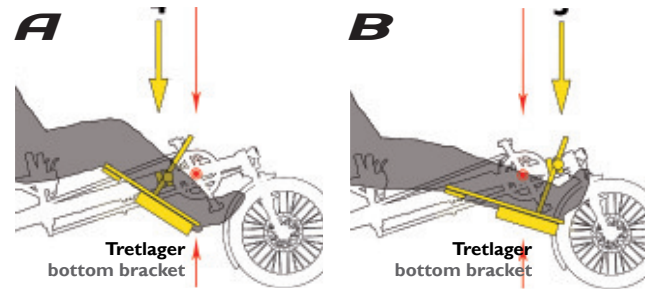
**Assembly for right side KETTWIESEL**

**A: rear bottom bracket** (tilted leg position)  
**B: ahead of bottom bracket** (straight leg position)

**Attention:** For fitting the leg rest on the right side you have to saw off the right crank handle.

**A:** To achieve a tilted leg position, you have to remove the frame quick-adjustment and shorten the chain (only KETT-WIESEL). After that, frame length-adjustment without changing the chain length won't be possible anymore.

- Unscrew the plastic front mount of the quick frame-adjustment. Loosen the right crank with use of a puller. Also loosen the chain by pushing the lower part of the adjustment backwards and lift the chain over the crank handle. (Fig. 1)
- Cut the crank handles as seen in fig. 2 (Place the cut in the highlighted area) and afterwards debourr the edges with a file. Re-mount the remaining gear.
- (For position A only:** Open the chain and remove the quick frame-adjustment (Fig. 3))
- Proceed by following steps 2,3 and 4 of the “assembly for left side”. (Turn the page)

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### Push rod Trets

The push rod is suited as a moving aid and to control the course of your child. The push rod is high adjustable and easy to mount.

#### Installation

##### 1. Figure A

Take off the seat cover and fit the W-4 38 clamps (1) with the rubber element around the backrest. Tighten the gusset (2) with M6x40 screws at the clamps by using a 5mm allen key. Place 4 spacers underneath at both sides. Fit the seat cover. Cut two holes into the seat cover for the M5x50 screws.

##### 2. Figure B

Fit the small clamps (3) around cross bar. Mount one gripper clamp (4) with M5x50 screws at the gusset and the other one with M5x45 screws at the small clamps. Put the bush (5) between the gripper clamps and tighten the screws by using a 4mm allen key.

3. Put the alu clamp (6, not displayed) on the bush. Clamp the push rod (7) at the desired high.

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## Long-term storage and disposal

### Preparing the Trets Reh for long-term storage, and for use after long-term storage

If you plan to store the Trets Reh for a longer period of time, e.g. during the winter months, then prepare the trike for storage as follows:

1. Clean the Trets Reh as described in the section “Cleaning your Trets Reh” on page 22.
2. Lubricate or wax the chain.
3. Tires gradually lose air when in storage. Storing the Trets Reh on flat tires for an extended period may damage the tires. Therefore, it is advisable to hang the Trets Reh so that the tires are not in contact with the floor. Otherwise, be sure to check the tire pressure of the stored trike regularly.
4. Store the trike in its lowest gear. This ensures that the gear cable is under minimal tension.
5. Store the Trets Reh in a dry and dust-free room. It may be wise to cover the trike with a tarp or similar dust cover.
6. Before using a Trets that has been in storage, be sure to check that the trike is in perfect working order, and have your authorized retailer perform a maintenance inspection.

### Preparing the Trets Reh for disposal

The construction of the Trets Reh is extremely durable and robust. However, if the trike must be disposed of, e.g., owing to irreparable damage following an accident, then contact your authorized retailer, who will help you dispose of the Trets Reh, or individual parts, in an environmentally sound manner. Furthermore, be sure to dispose of any replaced parts – such as tires, chains, brake pads, etc. – in an environmentally acceptable manner. In case of doubt, contact your authorized retailer.

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## Conditions of warranty and liability

The terms and conditions are based on the statutory warranty. This excludes parts that wear, such as the chain, tires, brakes, etc. The warranty period (statutory) of 24 months commences from the date of purchase. During this 2-year period, the dealer from whom the Hase Bikes trike was purchased is obligated to ensure that the vehicle has no fault that could destroy or significantly reduce its value or suitability for use. Therefore, the Hase Bikes trike must be in a safe condition and fit for use when handed over by the dealer. In addition, the company Hase Bikes offers a 3-year warranty (from date of purchase) against breakage of the frame in accordance with the following conditions. By sending in the enclosed registration card, you can extend this warranty period to 5 years.

- Hase Bikes will only be held liable for defects resulting from faulty frame manufacturing and assembly. For other assembly parts, the respective manufacturer is liable.
- This additional warranty only applies to the original owner and to usage in accordance with the user's manual.
- Within the warranty period of 3 years, Hase Bikes will repair, or if necessary replace, any broken frame part free of charge. Any defective parts replaced by Hase Bikes become the property of Hase Bikes.
- Any further claims beyond the scope of this warranty are excluded. Hase Bikes will not cover the costs of assembly or disassembly (e.g., by the retailer) or shipping. We will not accept postal deliveries with insufficient postage.
- The maximum permissible load (rider and luggage) of 65kg (143 lbs) specified for the Trets Reh – or 45 kg (99 lbs) when used as a trailer – may not be exceeded.
- If the Trets Reh is used in competition, ridden on rough terrain, or subjected to abnormally excessive stress, the statutory and additional warranty becomes void.
- Damage caused by accidents is not covered by the warranty.
- The warranty becomes void if the Trets Reh has been fitted with incompatible accessories or if accessories are installed incorrectly. Therefore, always have components replaced and installed by an authorized retailer.
- The warranty becomes void if improper modifications or alterations (e.g., grinding, drilling, bending) are made to the frame.
- Repairs or replacements made during the term of this warranty do not extend or renew the warranty term.
- Warranty claims can only be handled by the authorized Hase Bikes retailer from whom the trike was purchased or by one of our distribution partners.
- Children may only ride the Trets Reh under parental supervision.

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Have your vehicle inspected by your bicycle dealer at regular intervals. During the high season (March through September), it is best to schedule any required repairs or inspections in advance in order to avoid unnecessarily long downtimes and waiting periods.

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**Checklist:**

- Check pedals for tightness
- Check headset
- Check handlebar for tightness
- Check front and rear brakes
- Check front and rear wheel bearing
- Check bottom bracket
- Check cranks for tightness
- Check seat for tightness
- Check chain tension and wear
- Check wheels and air pressure
- Check rims and spokes
- Check all screws, bolts and nuts for tightness
- Check gear system
- Check lighting system
- Check accessoires

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